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Additional elements ≤ 1 %: Co

Contains no beryllium

remanium® CS+ must not be used for patients suffering from nickel allergy.

**Technical data**

Yield strength*	R <sub>0,2</sub>	340 MPa
Tensile strength*	R <sub>m</sub>	580 MPa
Hardness*	HV10	195
Fracture elongation*	A <sub>5</sub>	15 %
Modulus of elasticity*	E	170 GPa
Density		8.2 g/cm <sup>3</sup>
Solidus		1260 °C/2300 °F
Liquidus		1350 °C/2462 °F
CTE (25–500 °C/77–932 °F)		14 x 10 <sup>-6</sup> K <sup>-1</sup>
Color		white
Weight per ingot		approx. 5 g

\* Variations of +/- 10% are possible depending on the batch.

Use	NiCrMo bonding alloy compatible with conventional ceramic materials with appropriate CTE range
Melting	Torch, Induction (HF), Light arc Yes
Mirror like melt surface (HF) Casting time point (HF)	As soon as the molten mass begins to split
Pressure casting	250 – 450 mbar Observe instructions for use REF 989-448-00!
Welding	Possible using methods such as laser welding and TIG impulse welding
Filler material	NiCr welding wire, REF 528-220-00 (0.5 mm)
Presoldering Flux	NiCr Sold 1, REF 102-302-00 rema®-Flux 1, REF 102-304-00
Recommended ceramic	Ceramics with suitable CTE range such as ceraMotion® Me, Carmen® and CCS, from Dentaaurum
Cooling of ceramics in the furnace	Regular cooling, unless otherwise recommended by the ceramic manufacturer

**For further information see:****remanium®**, non-precious metal alloys for crown and bridgework, instructions for use **REF 989-448-00**Available free of charge from Dentaaurum or on the internet at [www.dentaaurum.com](http://www.dentaaurum.com)**Technical Hotline:****+49 72 31 / 803 -410****remanium®**  
**CS+**

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