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Dentaurum GmbH & Co. KG · Turnstr. 31 · 75228 Ispringen · Germany

2. Product description

discovery® smart sl is a self-ligating bracket manufactured using the MIM (Metal Injection Molding) process and features an integrated closure lid for the slot.

3. Intended purpose

For the treatment of tooth misalignments as part of orthodontic treatment.

4. Intended users

All products are intended for dental professionals only.

5. Indications

For tooth and jaw misalignments.

6. Contraindications

Do not use stainless steel brackets on patients with nickel allergies. Warnings are found on the respective product labels.

7. Patient target group

All patients for whom a fixed appliance with orthodontic braces can be placed as part of orthodontic treatment.

8. Processing

8.1. Informing the patient

Inform the patient about the possible consequences and risks of bracket treatment.

8.2. Conditioning teeth

Check each tooth thoroughly for any existing damage to the enamel to avoid damaging the tooth during debonding.

8.3. Selection of bracket adhesive

Dentaurum brackets can be bonded using all commercially available bracket adhesives.

8.4. Removal and handling

Grip the discovery® smart sl brackets mesially-distally using crimping tweezers to remove them from the packaging. Do not touch the laser-structured base of the brackets with your fingers and avoid any form of contamination, as this can negatively affect adhesion.

8.5. Using the bracket adhesive

Prepare the bracket adhesive according to the manufacturer's instructions. Apply a sufficient amount to the center of the bracket base. This guarantees optimum adhesion.

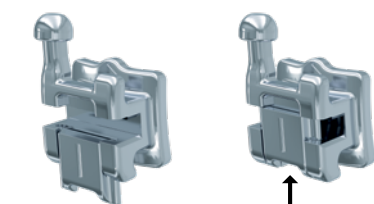
Fig. 1

Fig. 2

8.6. Placing and aligning brackets

After applying the adhesive, place the bracket onto the tooth in the usual way. The vertical center marking on the bracket lid helps orientation (Fig. 1).

8.7. Using the closure mechanism/ligating the archwires

The brackets are supplied with the lid open. For opening, we recommend the sl-instrument (REF 023-274-00). To close the bracket, push the lid toward the gingiva while applying light pressure with your finger until it clicks into place perceptibly (Fig. 2).

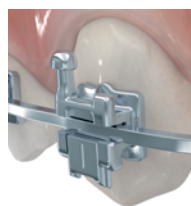
Fig. 3

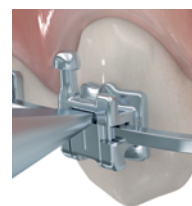
Fig. 4

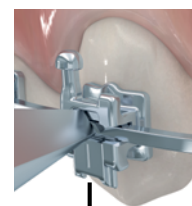
Fig. 5

All lids, both in the maxilla and in the mandible, are opened from gingival to incisal or occlusal (Fig. 3). To open the lid, insert the pointed end of the sl-instrument into the slot on the top section of the bracket (Fig. 4). The lid can be easily opened with a slight twisting motion using the sl-instrument (Fig. 5).

8.8. Bracket debonding

Weingart pliers can be used to debond discovery® smart sl brackets. Grasp the bracket from mesial-distal on the tie wings. Turn the bracket briefly to the left or right, without pulling, to remove it from the tooth. Alternatively, you can also use the bracket removal pliers REF 003-349-00, which allow you to trim the bracket between the bracket base and the adhesive.

9. Composition

The discovery® smart sl brackets are made of the following materials: DIN, 1.4404, 1.4542 and 1.4310. The composition is given in the orthodontic materials list; please refer to the catalog or www.dentaurum.com. The SSCP is available at <https://ec.europa.eu/tools/eudamed> and www.dentaurum.com.

10. Safety information

The product should not be used if there is a known allergic reaction to one or more of the material components. Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, they should inform the manufacturer and the competent authority in the country in which they are resident.

11. Storage and shelf life

Store discovery® smart sl brackets in their closed packaging. Before use, check the product for integrity and cleanliness; dispose of it if necessary.

12. Disposal

When disposing of the product, observe the applicable national regulations and the information in the safety data sheets, if available.

13. Products for single use

The product is intended for single use only. Reconditioning of a product that has already been used (recycling) as well as its reuse on a patient is not permitted.

14. Product range

For the complete product range of discovery® smart sl brackets, please see our orthodontics catalog or www.dentaurum.com.

15. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as Dentaurum has no influence on the processing on site.

16. Explanation of symbols used on the label

Refer to the label. Additional information as well as the respective Instructions for use can be found on the internet at www.dentaurum.com (explanation of the symbols used on the label REF 989-313-00).

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