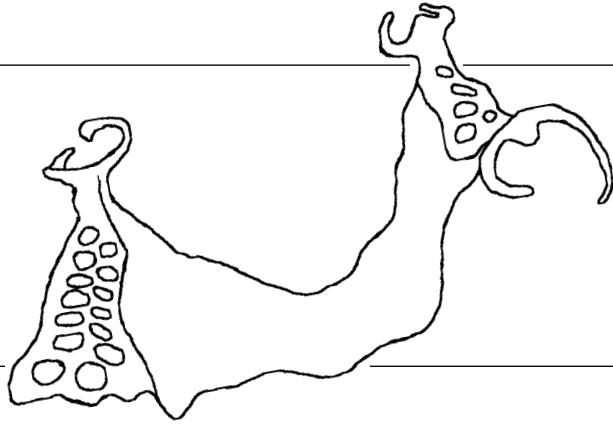
Gebrauchsanweisung | Instructions for use |  
Mode d'emploi | Modo de empleo | Modalità d'uso

rema<sup>®</sup>  
**Exakt F R**

Phosphatgebundene Einbettmasse  
Phosphate bonded investment material  
Revêtement à liant phosphate  
Revestimiento ligado con fosfato  
Rivestimento a legante fosfatico

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## Dear customer

Thank you for choosing a quality product from Dentaorium.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at [www.dentaorium.com](http://www.dentaorium.com), even though you frequently use the same product.

## Manufacturer

Dentaorium GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

## Safety information

**Investments contain quartz. Avoid inhalation of dust as this may cause lung diseases (silicosis / lung cancer).**

**Wearing of face mask type FFP 2 – EN 149:2001 is recommended.**

**Open pouch with scissors and avoid dust development when pouring into mixing bowl.**

**Rinse empty pouch with water before disposal.**

**Remove dust from working area only with a wet cloth.**

**To avoid dust when devesting place ring into water until it is completely soaked after ring has reached room temperature.**

**When sandblasting use suction with fine dust filter.**

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EN

## Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

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# Dentaurum Group

Germany | Benelux | España | France | Italia | Switzerland | Australia | Canada | USA  
and in more than 130 countries worldwide.

DENTAURUM  
QUALITY  
WORLDWIDE  
UNIQUE

- Informationen zu Produkten und Serviceleistungen finden Sie unter [www.dentaurum.com](http://www.dentaurum.com)
- For more information on our products and services, please visit [www.dentaurum.com](http://www.dentaurum.com)
- Vous trouverez toutes les informations sur nos produits et services sur [www.dentaurum.com](http://www.dentaurum.com)
- Descubra nuestros productos y servicios en [www.dentaurum.com](http://www.dentaurum.com)
- Informazioni su prodotti e servizi sono disponibili nel sito [www.dentaurum.com](http://www.dentaurum.com)

Stand der Information | Date of information | Mise à jour |  
Fecha de la información | Data dell'informazione: 10/18

Änderungen vorbehalten | Subject to modifications | Sous réserve de modifications |  
Reservado el derecho de modificación | Con riserva di apportare modifiche

**D**  
DENTAURUM